

A photograph of a female doctor with dark hair and glasses, wearing a white lab coat, smiling warmly at an elderly female patient with white hair. The patient is resting her chin on her hand and looking down thoughtfully. The image is overlaid with a semi-transparent dark blue filter.

# Here's What You Need To Get Started with **Kallisio**



Treat with precision. Personalize every therapy.



# welcome to **kallisia**



## **kallisia**

### WHO WE ARE



**WE ARE DEDICATED TO  
REVOLUTIONIZING HEAD AND NECK  
RADIATION THERAPY WITH INNOVATIVE,  
PATIENT-SPECIFIC SOLUTIONS.**

### WHAT WE OFFER



**STENTRA™, A PATIENT-SPECIFIC 3D  
PRINTED IMMOBILIZATION ORAL STENT  
MADE FOR PRECISION, COMFORT, AND  
HEALTHY TISSUE PRESERVATION.**

### WHY IT MATTERS




**STENTRA™ ENHANCES PATIENT SAFETY  
AND COMFORT WHILE EMPOWERING  
ONCOLOGISTS WITH PRECISION TOOLS  
TO IMPROVE TREATMENT OUTCOMES.**



# stentra™ patient-specific intraoral stent


INTENDED USE

**DISPLACE HEALTHY TISSUE  
IN RADIATION THERAPY**

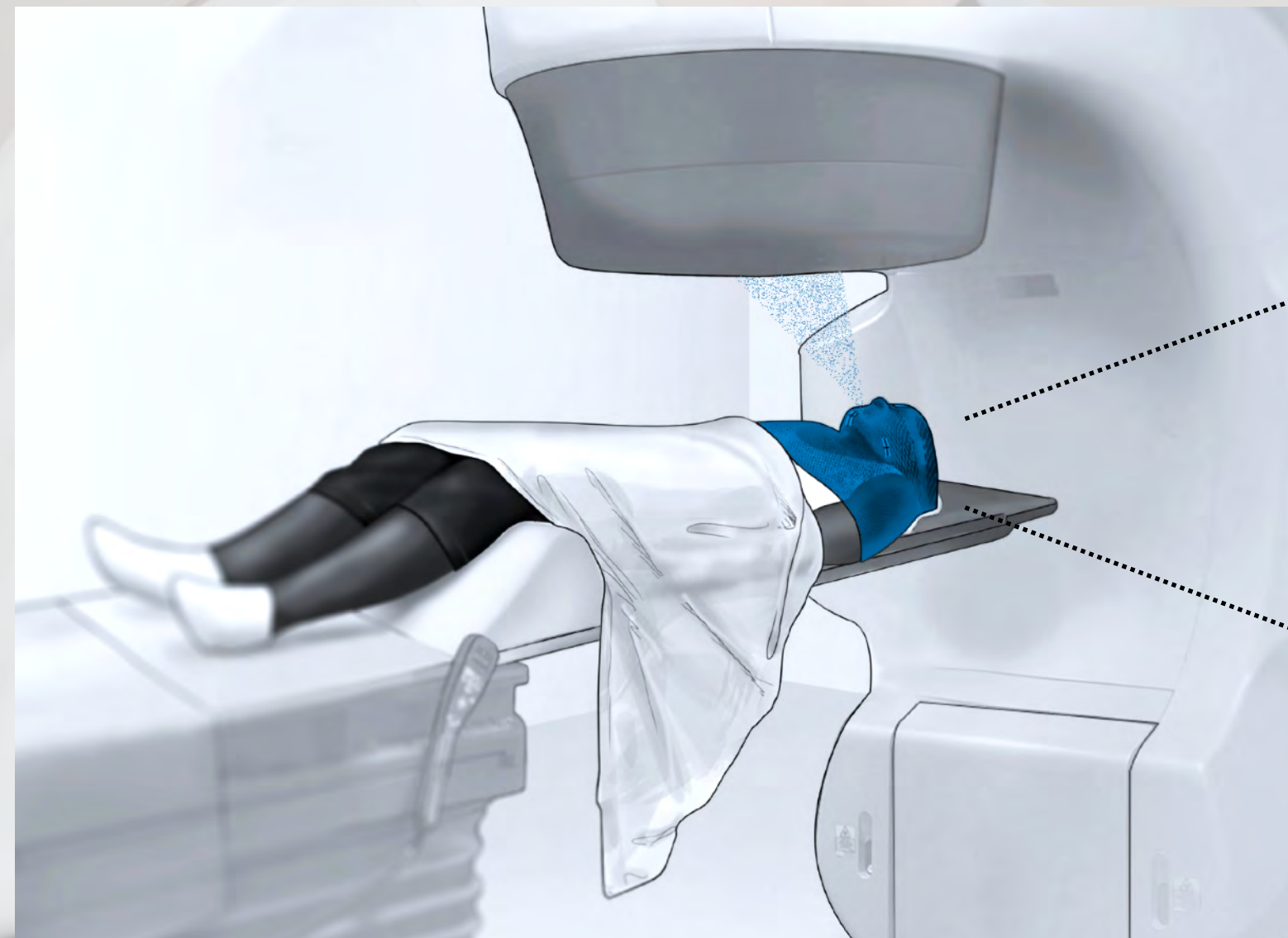

INDICATION

**PATIENT-SPECIFIC  
IMMOBILIZATION & MOUTH OPENING**

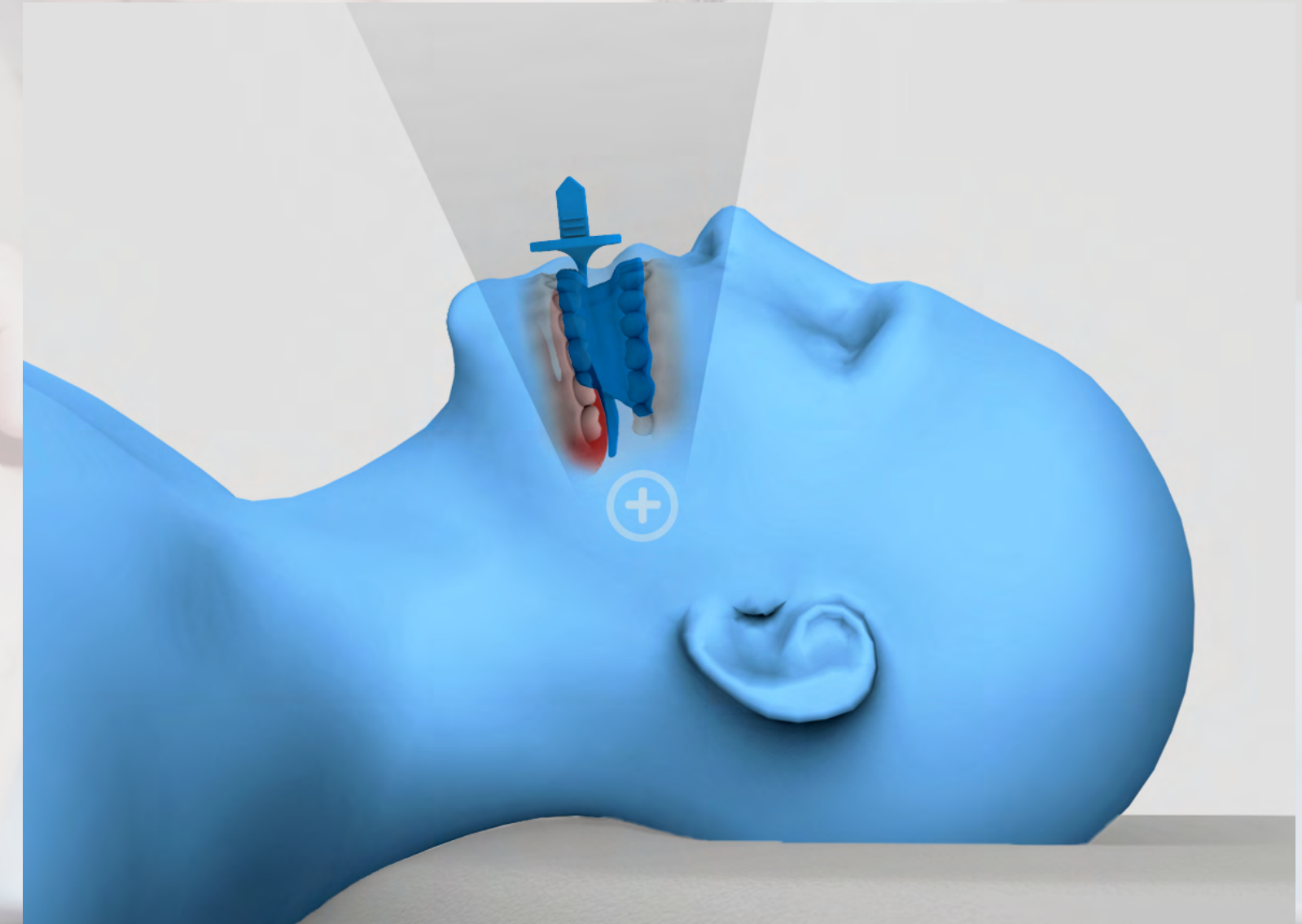



# lets identify **your needs**



?

ARE YOU LOOKING TO?

**REDUCE SETUP TIME & INCREASE  
PATIENT POSITIONING  
CONSISTENCY**

?

DO YOU NEED TO?

**BALANCE TUMOR CONTROL  
WITH HEALTHY TISSUE  
PRESERVATION**

?

ARE YOU SEEKING TO?

**STREAMLINE TREATMENT WITH  
PERSONALIZED PRECISION FOR  
EACH UNIQUE CASE**




# stentra™ system digital workflow

VISIT 1 (CONSULT)  
SCAN & Rx ORDER

KALLISIO  
PROVEN AUTOMATION

VISIT 2 (SIM)  
VERIFY DEVICE

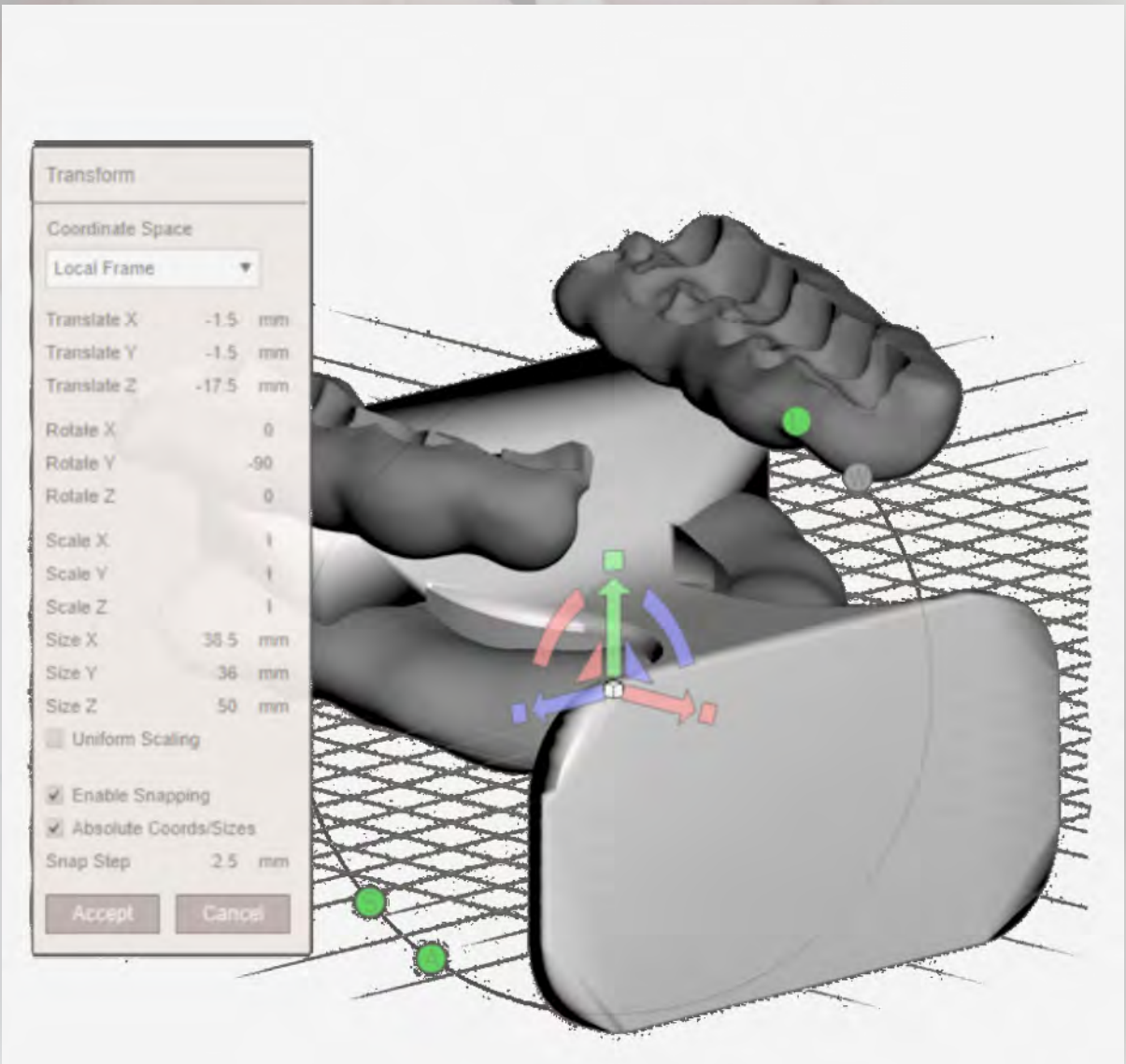

1

Intraoral Scan

2

Device Design

3

3D Printed Device

4

Radiation Simulation



72 hrs  
turnaround



**Performance & Portability**


1

2

**Proprietary Mouth Opening Bite Block**

3

**15 minute Intraoral Scan**

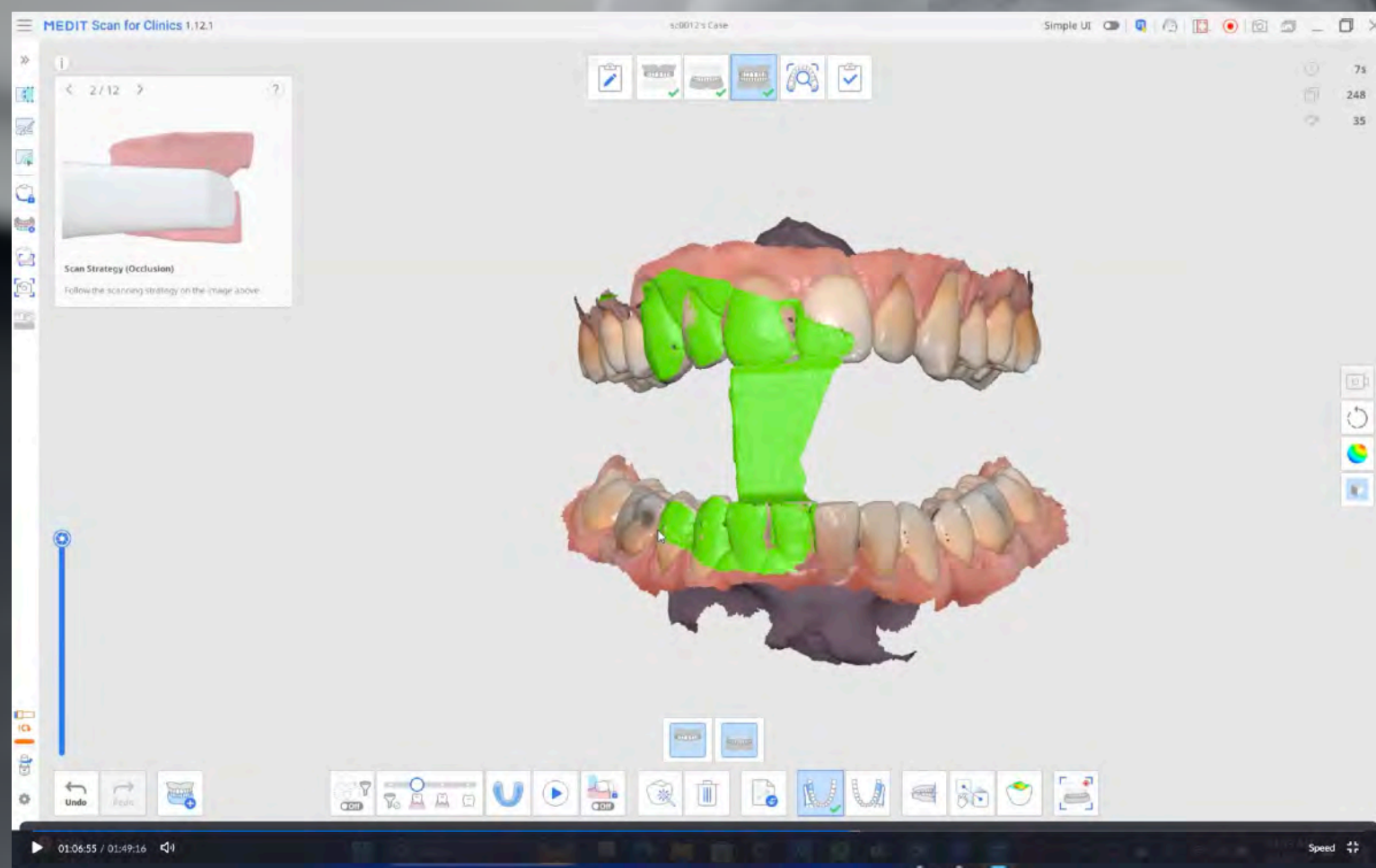

**Intraoral Scan**



# stentra™ personnel & facilities

VISIT 1  
ORDER DEVICE


used by **oncology team**  
**or dental specialist** during  
treatment planning visit at **radonc offices**

VISIT 1  
PERFORM SCAN

performed by **oncology team**  
**or dental specialist** during  
treatment planning visit at **radonc offices**

VISIT 2  
VERIFY DEVICE

performed by **radiation oncologist**  
during simulation visit at  
**radiation treatment room**

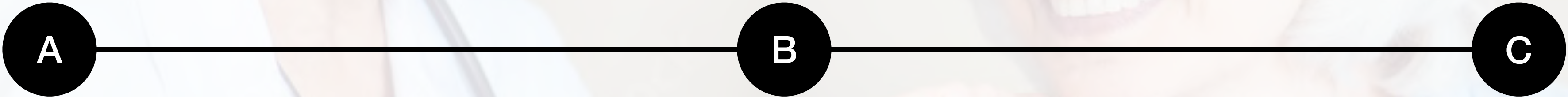

oncology team



dental specialist




oncology team



dental specialist



radiation oncologist

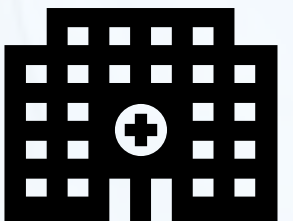



Reprocessing  
Container

1

3 Patient-Specific  
Oral Stent

What's In The Box

2

Factory Sealed  
Polybag

**Stentra**  
mouth opening immobilizing oral stent






# stentra™ system workflow tools

## VISIT 1 ORDER DEVICE

Use **kalliso portal** to generate UDID to track each patient case and capture intraoral scan



## VISIT 2 VERIFY DEVICE

Access **kalliso portal** during simulation to verify that patient-specific stentra™ immobilizes and fits patient

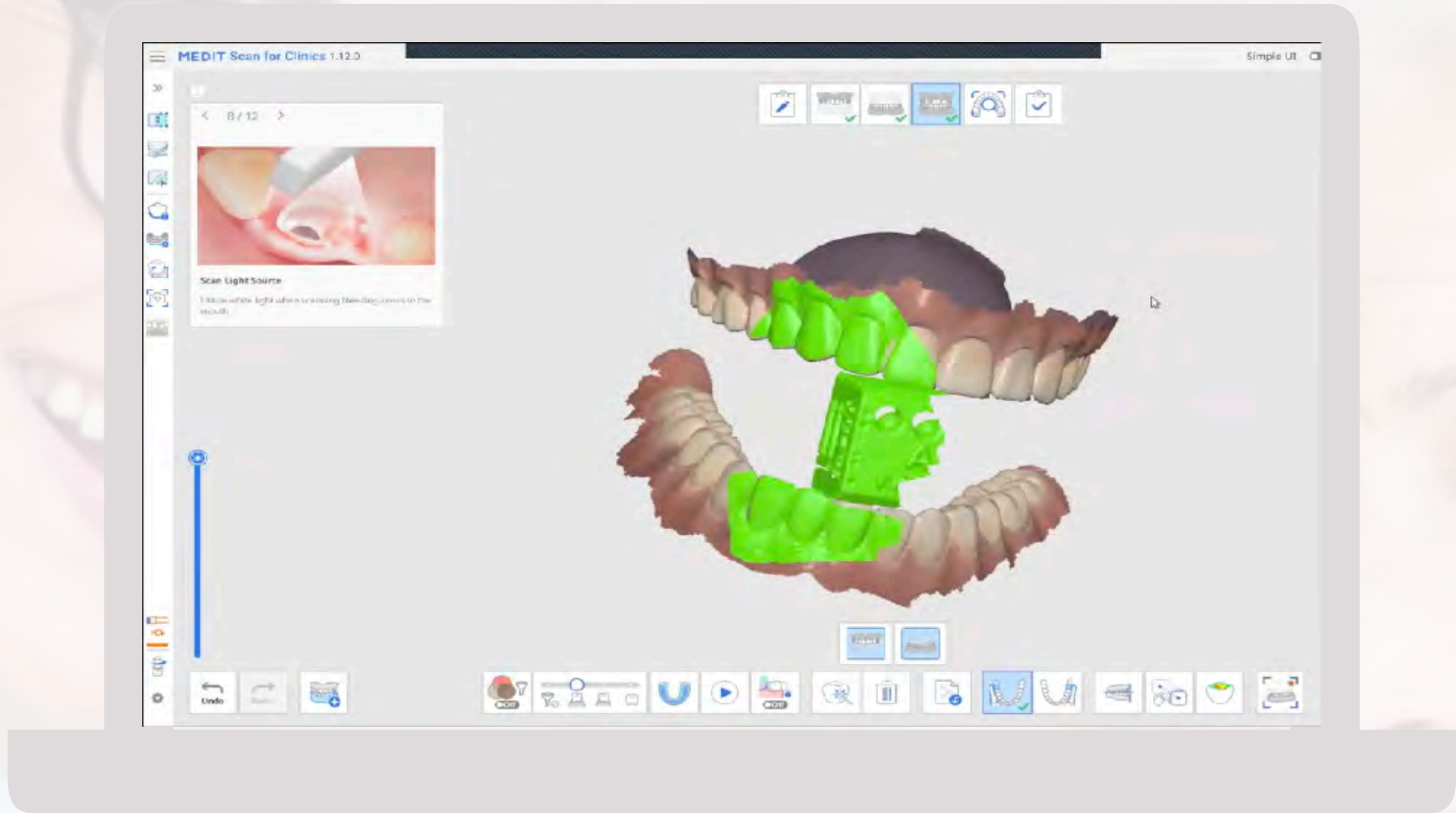

## VISIT 1 CAPTURE SCANS

Access **scanner software** to capture patient intraoral scan using the optical scanner and kalliso provided **qbite bite block**

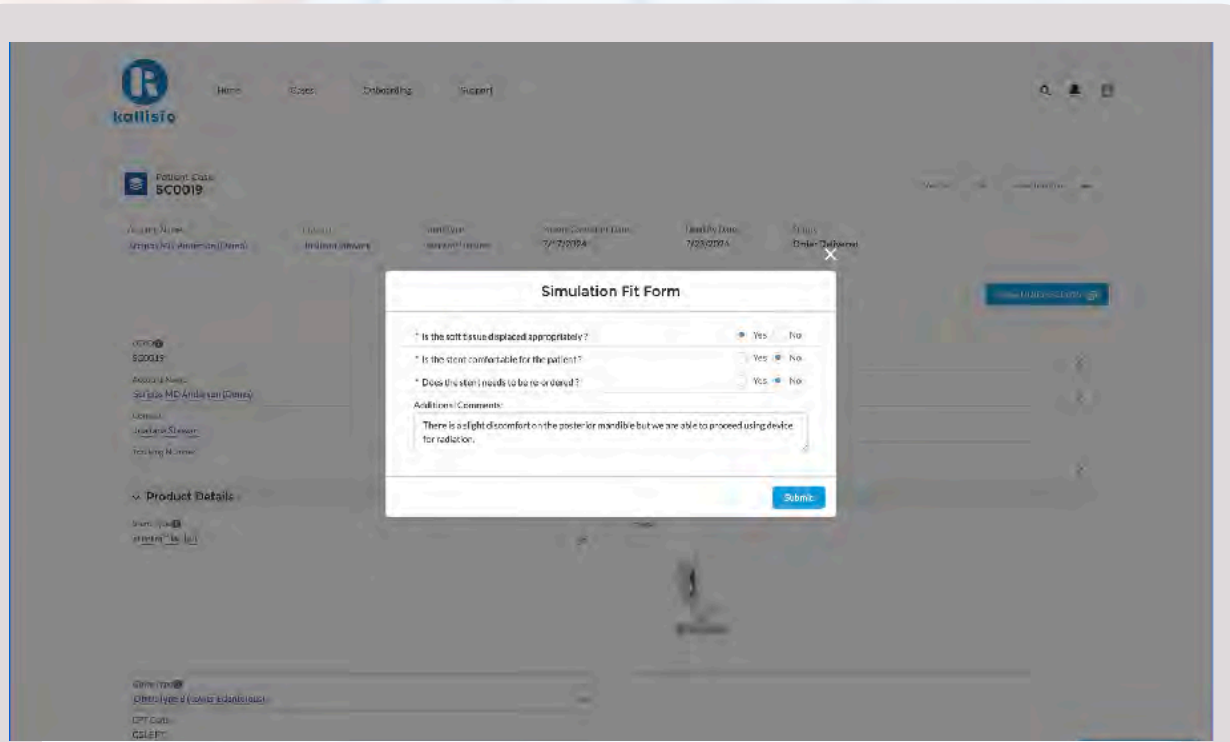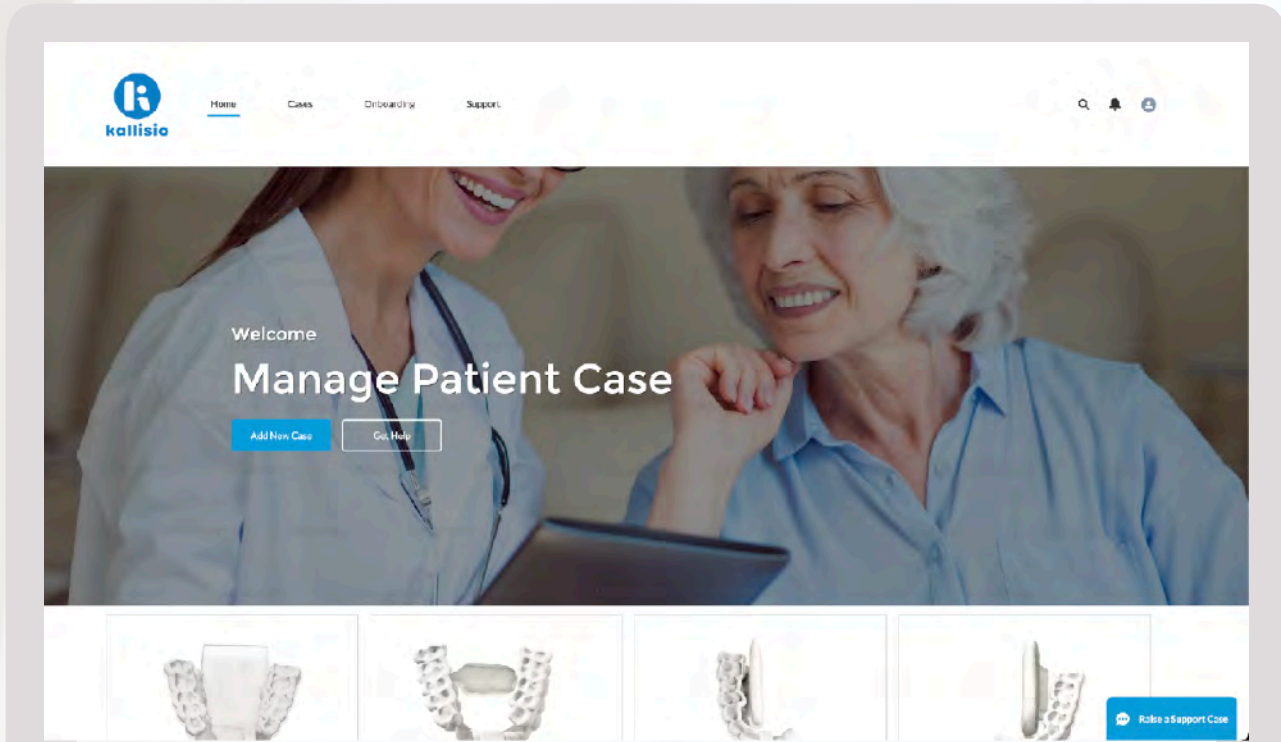



[Home](#)[Cases](#)[Onboarding](#)[Support](#)

## Create a Case

\* Stent Type

stentra™ lat-left [\$650.00]

\* Qbite Type

Qbite Type A (Dentulous) [\$10.00]

☒ Use Default Account Shipping Address

\* Shipping Address

\* Shipping Street

10670 John Jay Hopkins Drive

\* Shipping City

San Diego

\* Shipping State/Province

CA

\* Shipping Zip/Postal Code

92121

\* Shipping Country

United States

Description


Please deliver these to Radiation Room #1010

\* Need By Date

8/5/2024

Ship To Contact

Rajan Patel

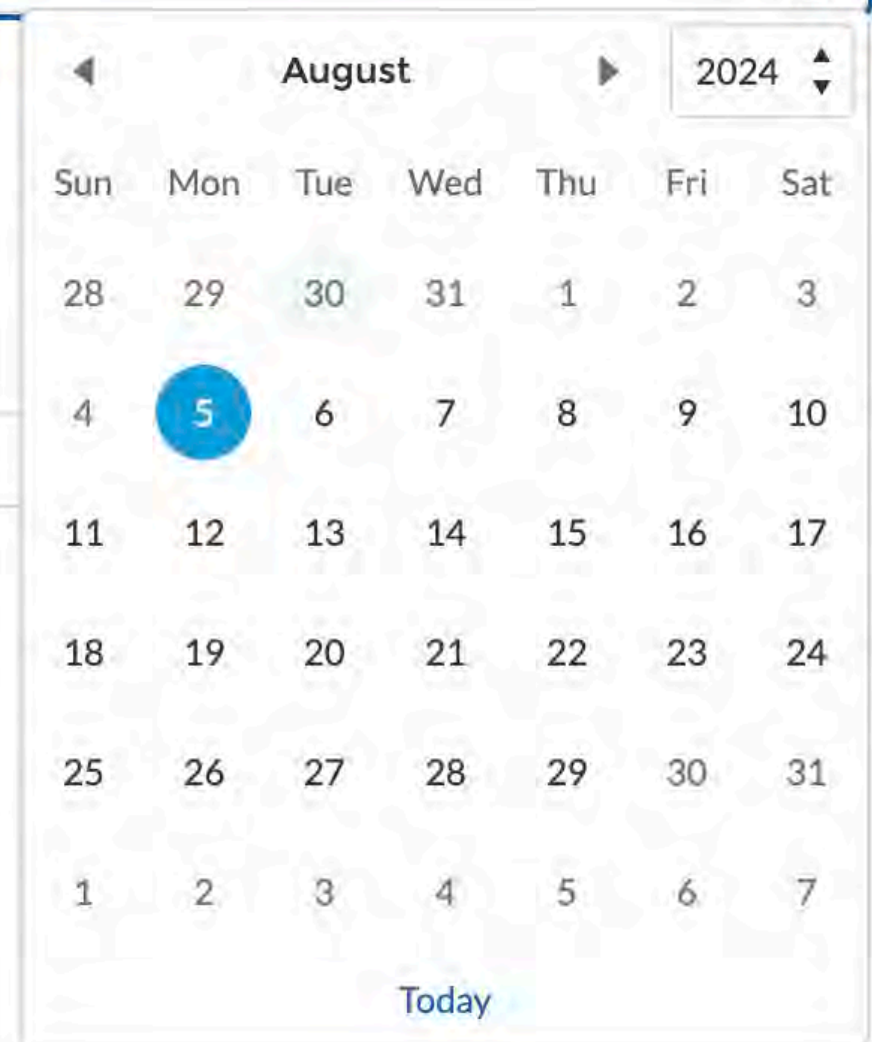

Reset

Review

Copyright 2023 kallisio Inc - Focused on eradicating toxicity in cancer care. | [Privacy Policy](#) | [Terms & Conditions](#)

# CLINICAL PORTAL





Patient Cases

Recently Viewed ▼



28 items • Updated 3 minutes ago

	UDID	Stent Type	Patient Case Start Date	Need By Date	Status	
1	SC001B	stentra™ lat-left	7/30/2024	8/5/2024	Draft	
2	SC001C	stentra™ elevate	7/30/2024	8/5/2024	Draft	
3	SC001A	stentra™ lat-right	7/17/2024	7/23/2024	Stent Verified	
4	SC0019	stentra™ lat-left	7/17/2024	7/23/2024	Sent for Manufacturing	
5	SC0018	stentra™ lat-right	7/16/2024	7/22/2024	Sent for Manufacturing	
6	SC0017	stentra™ depress	7/15/2024	7/19/2024	Manufacturing In-Progress	
7	SC0016	stentra™ depress	7/15/2024	7/19/2024	Manufacturing In-Progress	
8	SC0015	stentra™ lat-right	6/24/2024	6/28/2024	Draft	
9	SC0014	stentra™ depress	6/24/2024	6/28/2024	Draft	
10	SC0013	stentra™ depress	6/24/2024	6/28/2024	Draft	
11	SC0012	stentra™ depress	6/24/2024	6/28/2024	Draft	▼
12	SC0011	stentra™ lat-left	6/24/2024	6/28/2024	Draft	▼
13	SC0010	stentra™ depress	6/23/2024	6/28/2024	Manufacturing In-Progress	▼
14	SC000Z	stentra™ depress	6/17/2024	6/28/2024	Manufacturing In-Progress	▼
15	SC000Y	stentra™ depress	6/17/2024	6/28/2024	Manufacturing In-Progress	▼
16	SC000X	stentra™ depress	6/3/2024	6/7/2024	Order Delivered	▼

Notifications

[Mark all as read](#)



Order Shipped

Your Order SC0018 has been shipped.

a few seconds ago •



Order Delivered

Your Order SC0019 has been delivered. Verify now!

a minute ago •



Order Shipped

Your Order SC0019 has been shipped.

a minute ago •



Order Delivered

Your Order SC001A has been delivered. Verify now!

Jul 17, 2024, 11:47 AM



Order Shipped

Your Order SC001A has been shipped. Your Tracking Number: SAMPLE12345

Jul 17, 2024, 11:47 AM •

ORDER TRACKING



Patient  
Comfort

1

2

Oral  
Immobilization

3

Fit with  
Radiation Mask


**STENTRA™ SIMULATION**



# data flow no PHI or EHR integration required



## VISIT 1 ORDER DEVICE

The **kallisio portal**  
**customer data** is hosted  
within a Salesforce Health Cloud instance  
**usa364, Oregon USA**  
Comply with HIPAA & GDPR standards

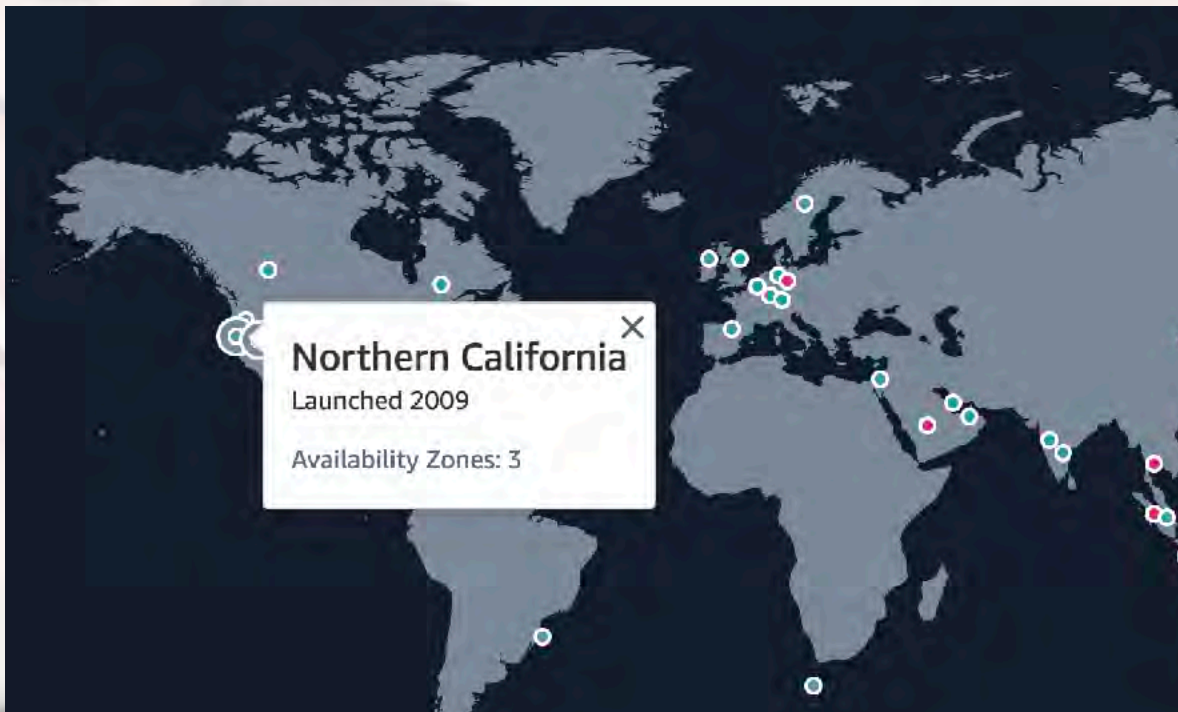


## VISIT 2 VERIFY DEVICE

The **kallisio portal**  
**customer data** is hosted  
within a Salesforce Health Cloud instance  
**usa364, Oregon USA**  
Comply with HIPAA & GDPR standards

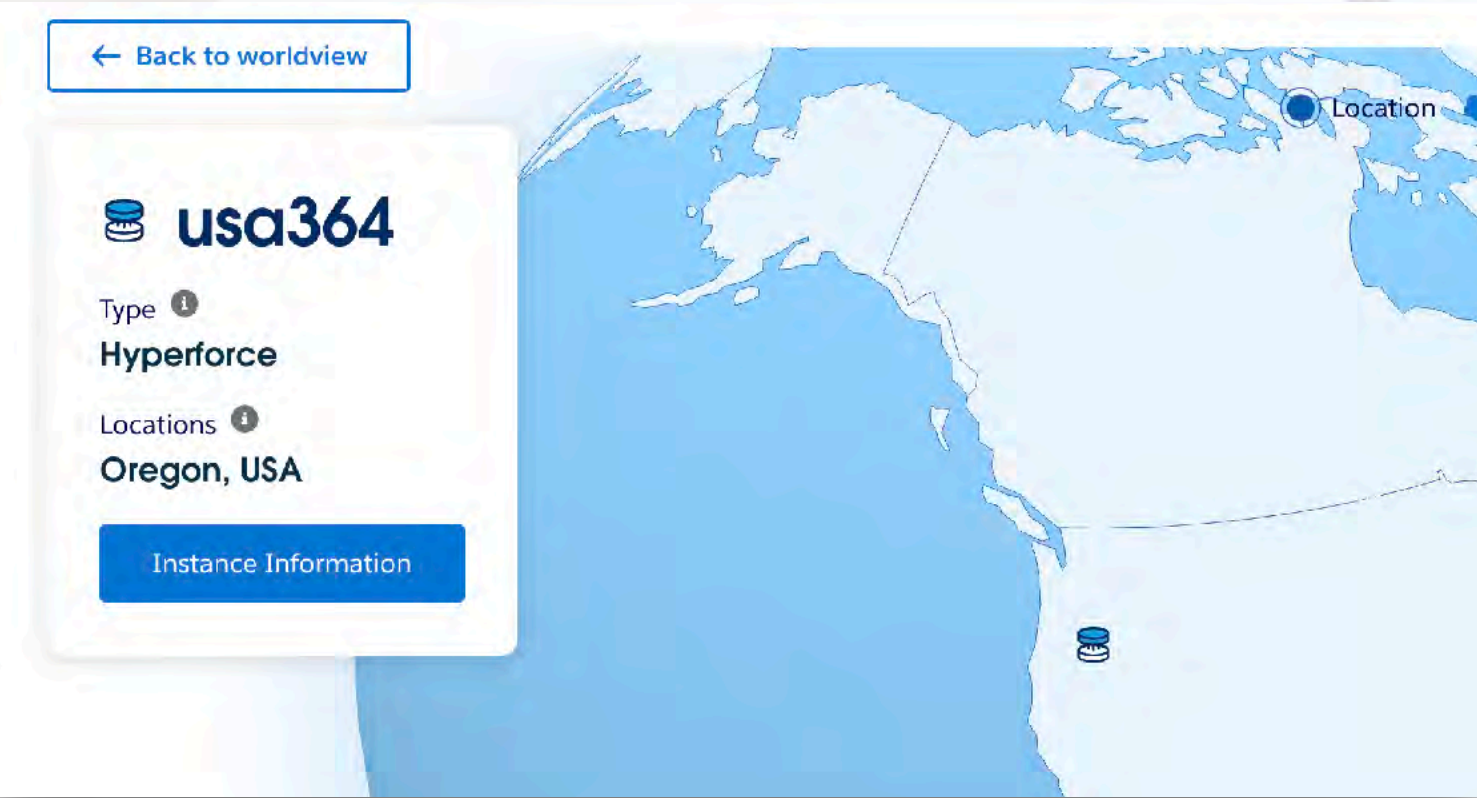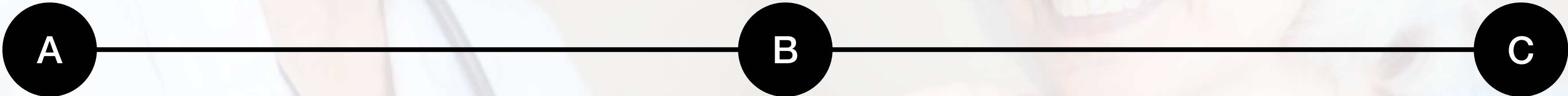

## VISIT 1 PERFORM SCAN

the **scanner software** scan data  
is stored with **kallisio Cloud** (AWS)  
and on your **local computer**.

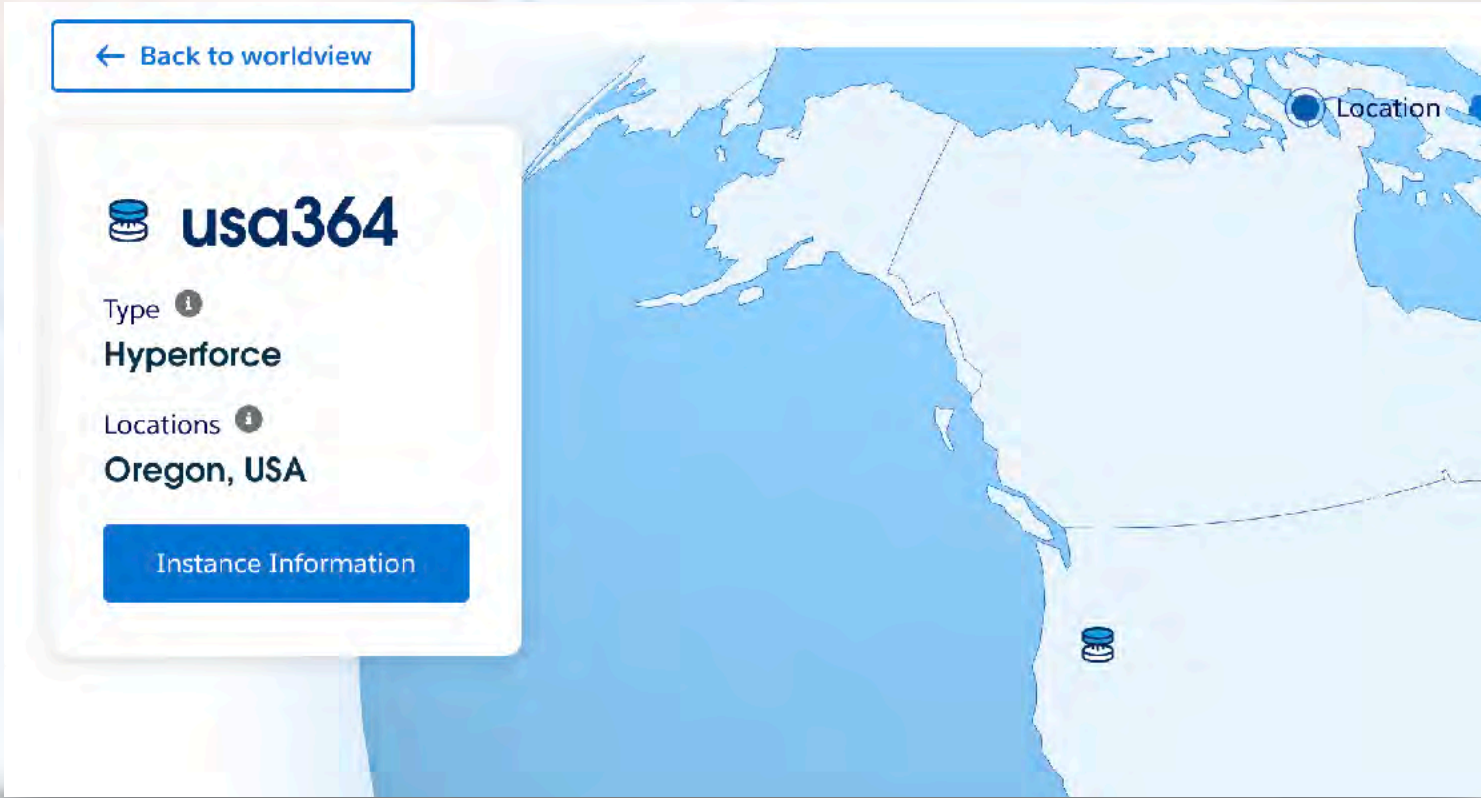



# stentra™ key benefits

**Improve**  
**Patient**  
**Outcomes**


**Reduce**  
**Radiation**  
**Dose**

**Reduce**  
**Patient**  
**Symptoms**





# summary what you need

## VISIT 1 & 3 KALLISIO PORTAL

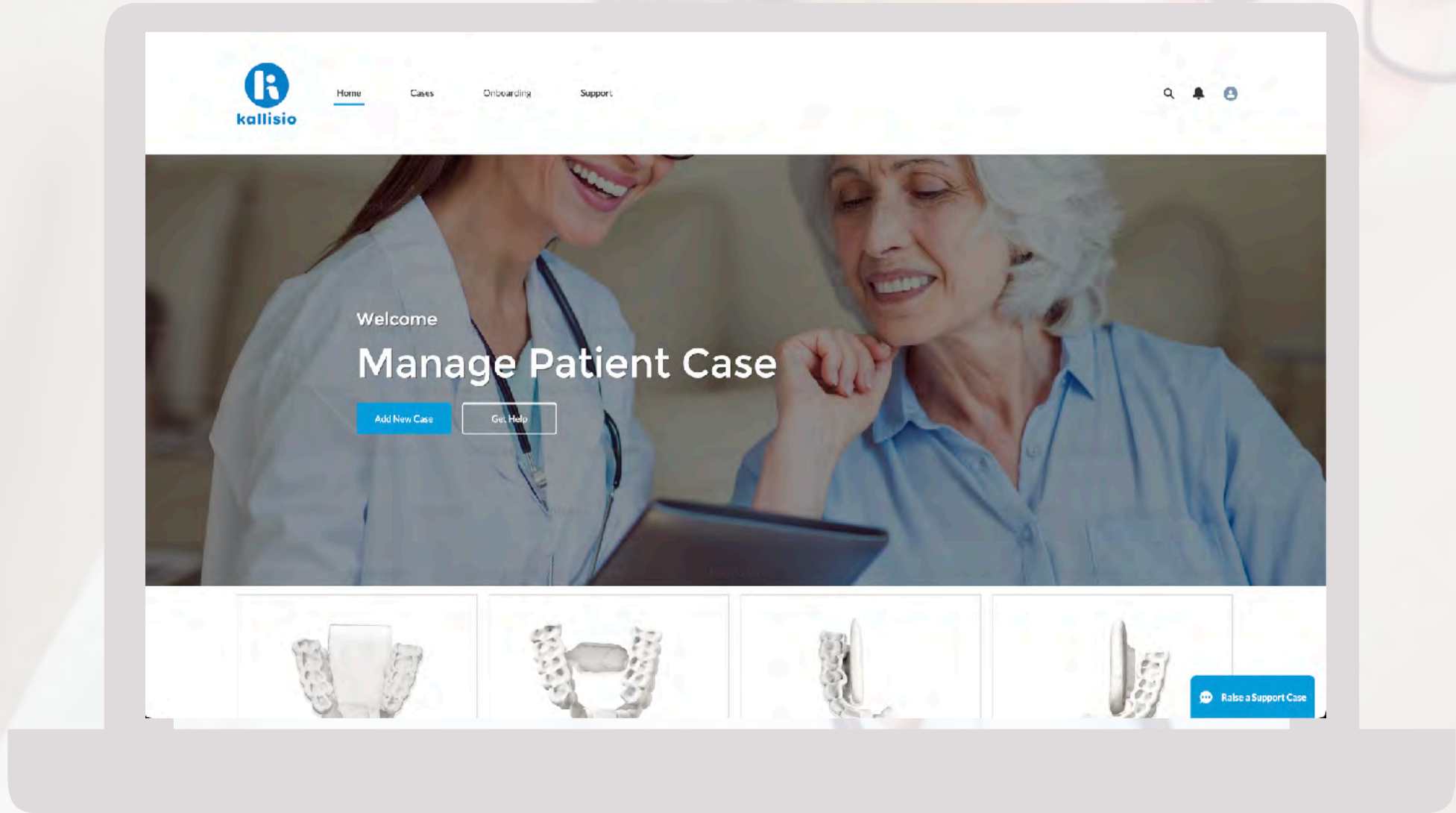


## VISIT 1 SCANNER & ACCESSORIES

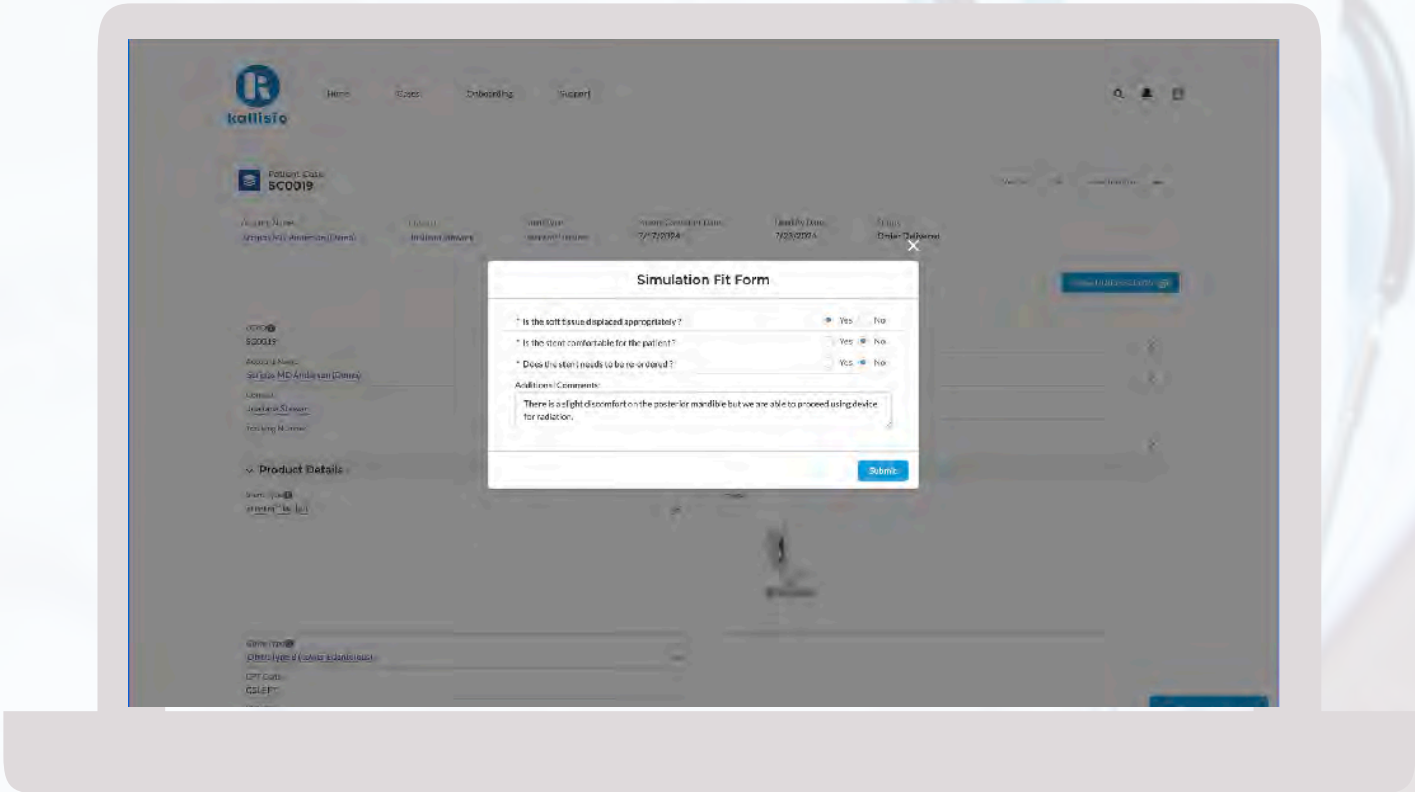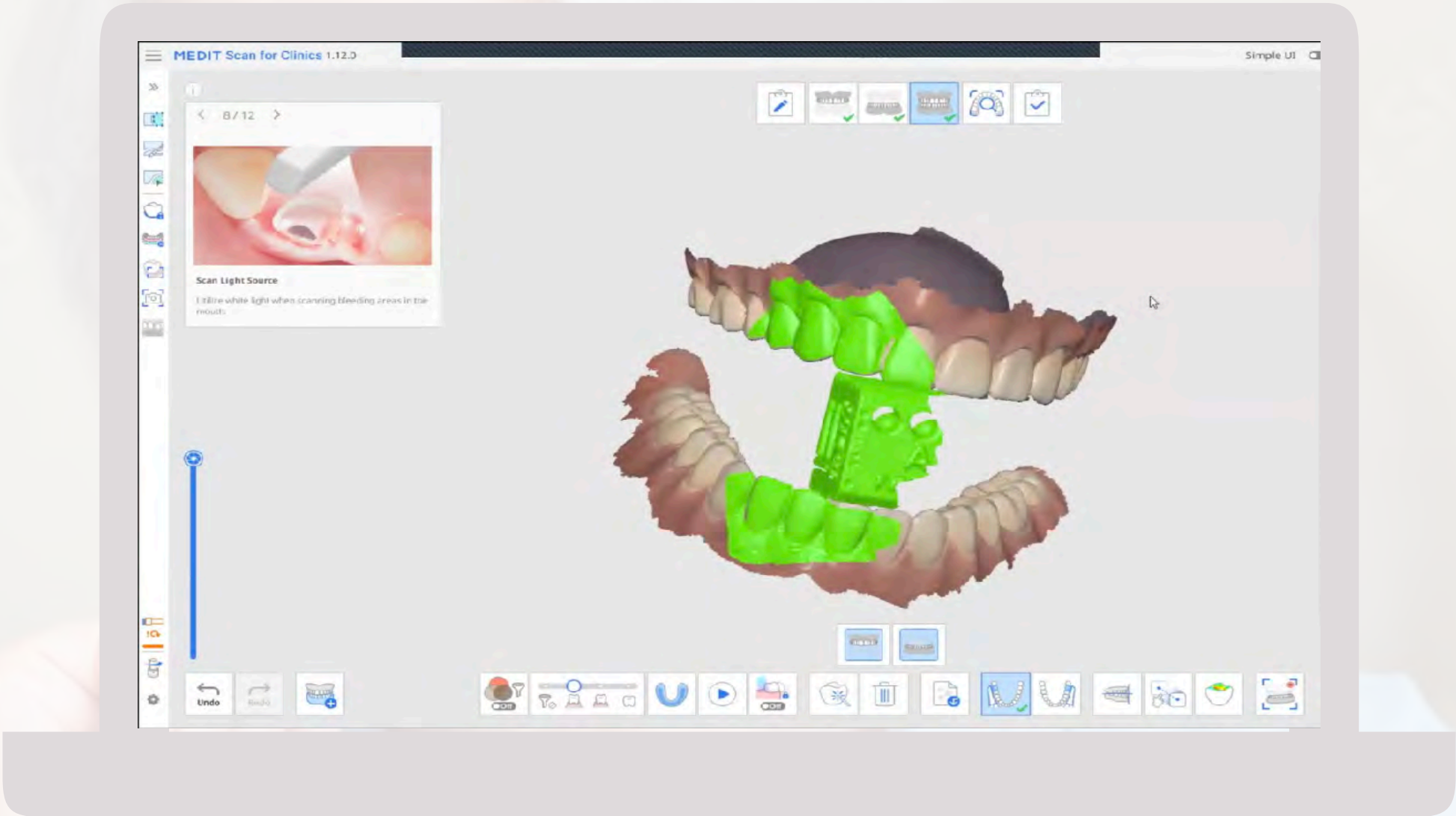

SIMULATION FORM



SCANNER + TIP

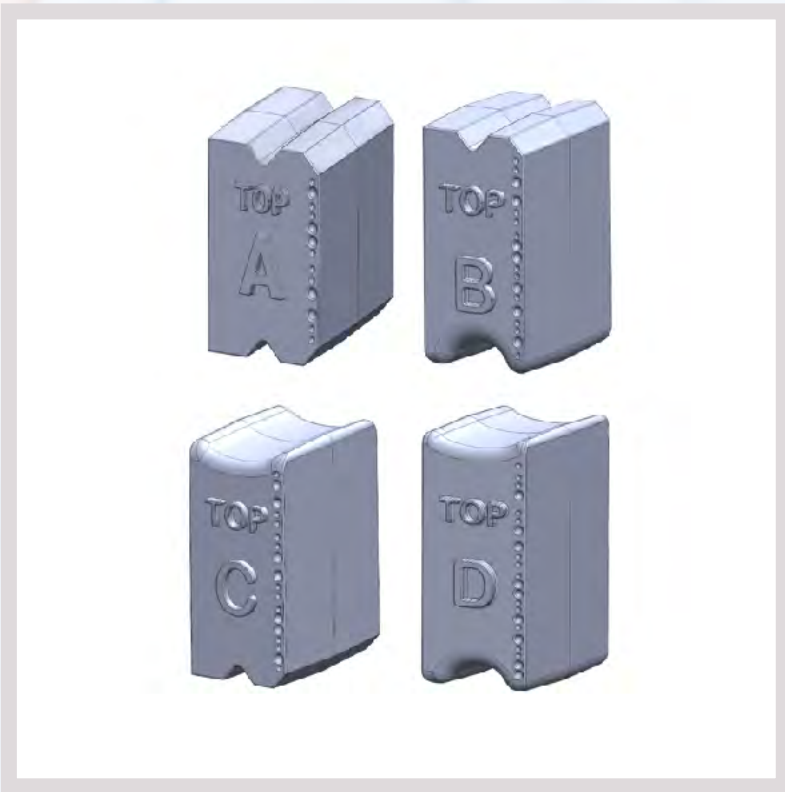

QBITES BITE BLOCK





**kallisio**

**contact us for consult**

**BOOK NOW**

**EMAIL [NICOLE@KALLISIO.COM](mailto:NICOLE@KALLISIO.COM)**

our team is here to support you every step of the way.